


CE-D09 Spare parts list



Identification number: TCF-VOMNL-2410-14C
Product group: Walking aid
Productname: Travixx Classic
Batchnumber:

Backrest

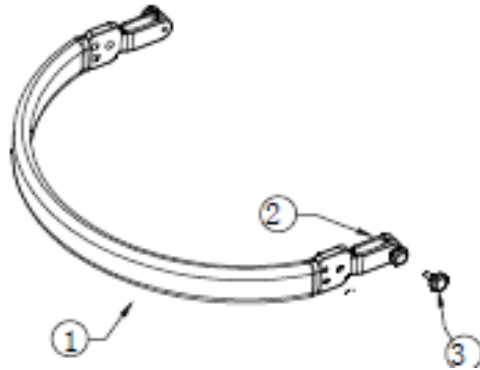


Drawing No.	Item No.	Description	Purchase Price
1	92415200	Backrest strap incl. detachable cover	€ 47,15
2	-	Not available as spare part	-
3	92415205	Star knob grey for backrest	€ 3,93

CE-D09 Spare parts list



Identification number: TCF-VOMNL-2410-14C
Product group: Walking aid
Productname: Travixx Classic
Batchnumber:

Bag

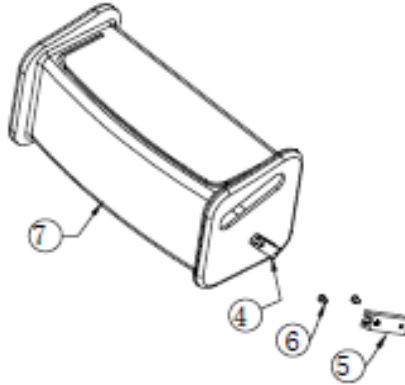


Drawing No.	Item No.	Description	Purchase Price
4 to 6	92429950	Mounting block (set of 2) for mounting the bag	€ 8,92
7	92419900	Bag, Front	€ 17,05

CE-D09 Spare parts list



Identification number: TCF-VOMNL-2410-14C
 Product group: Walking aid
 Productname: Travixx Classic
 Batchnumber:

Seat

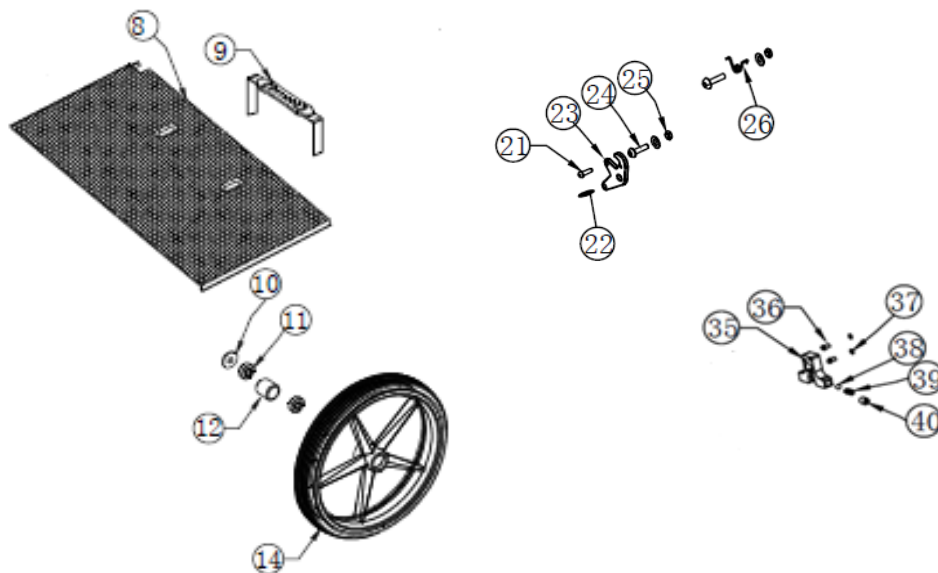


Drawing No.	Item No.	Description	Purchase Price
8	52411100	Wiremesh seat lightweight	€ 51,10
9	92419050	Unlocking belt for folding mechanism	€ 44,03
10 to 12	92417060	Bearing set for front wheels incl. mounting material front wheel	€ 16,12
	92417050	Cover caps for front wheel, Black (set of 2)	€ 3,93
14	82411100	Front wheel (large), Black spoked	€ 27,51
21 to 25; 35 to 40	92419055	Folding mechanism and locking mechanism	€ 44,03

CE-D09 Spare parts list



Identification number: TCF-VOMNL-2410-14C
 Product group: Walking aid
 Productname: Travixx Classic
 Batchnumber:

Frame assembly

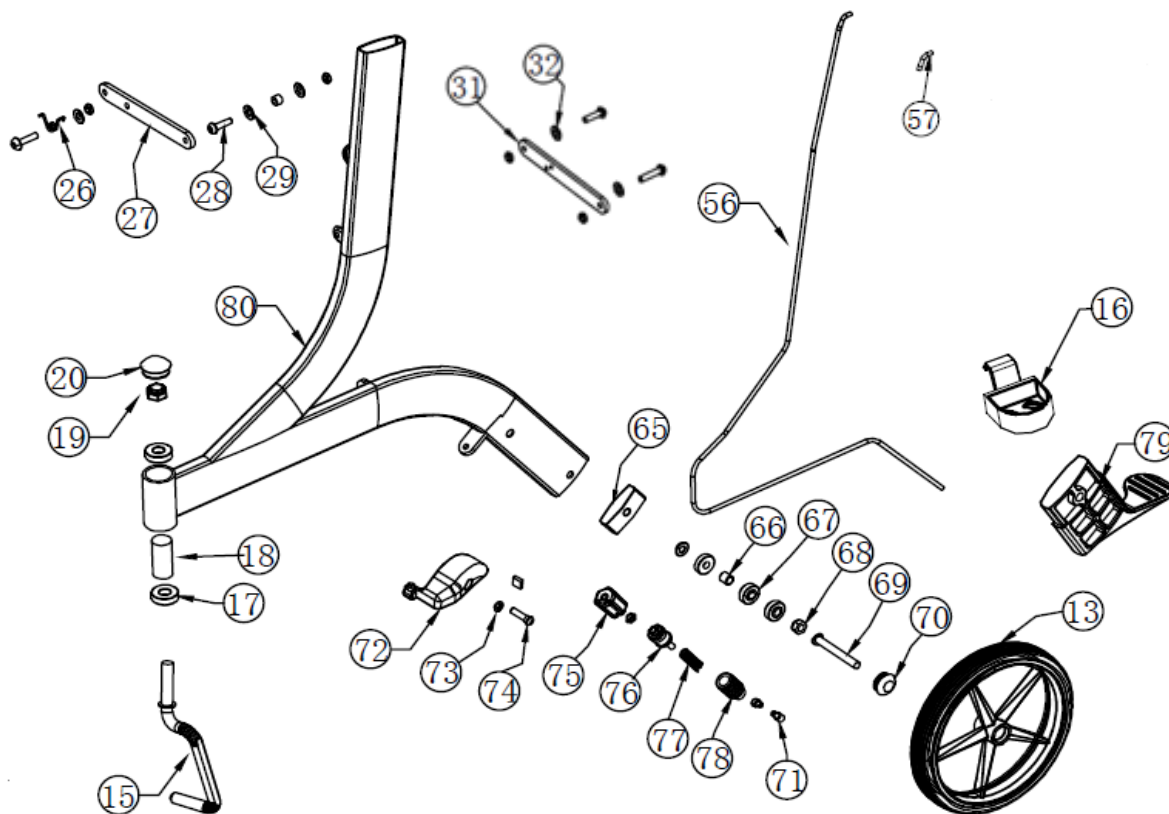


Drawing No.	Item No.	Description	Purchase Price
13	42411100	Rear wheel (small) Black spoked	€ 23,59
15	82410150	Front fork Left Black (thin)	€ 38,53
	82410155	Front fork Right Black (thin)	€ 38,53
16	92417125	Crutch holder and clamp	€ 13,76
17 to 19	-	Not available as spare part	-
20	92417035	Bearing house cover caps, set of 2	€ 3,93
27 to 29	-	Not available as spare part	-
31 to 32	-	Not available as spare part	-
56 + 57	92417100	Brake cable Travixx Small (1x inner- and outer cable)	€ 23,59
	92417105	Brake cable Travixx Medium (1x inner- and outer cable)	€ 23,59
65 to 69	-	Not available as spare part	-
70	ALPL04006	Cover cap for rear wheel (Set of 2)	€ 3,93
71 to 78	92417150	Brakepad incl. mounting material	€ 18,42
79	92417120	Stepper per pair (plastic)	€ 26,74
80	-	Not available as spare part	-

CE-D09 Spare parts list



Identification number: TCF-VOMNL-2410-14C
 Product group: Walking aid
 Productname: Travixx Classic
 Batchnumber:

Cross frame

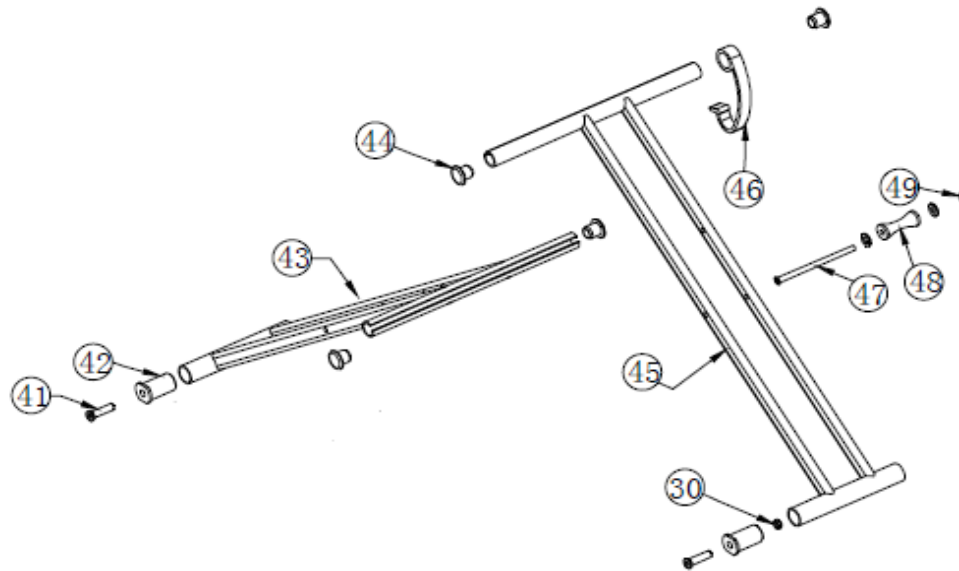


Drawing No.	Item No.	Description	Purchase Price
41 & 42	92419020	Bushings for crossframe/sideframe with bolts (set of 4)	€ 17,09
30; 41 to 49	92419250	Cross frame Travixx Classic S	€ 75,48
	92419150	Cross frame Travixx Classic M	€ 75,48
44	92419005	Cover caps for cross bar incl screws, set of 4	€ 11,79
46	92419100	Transport locking lever (grey)	€ 13,76

CE-D09 Spare parts list



Identification number: TCF-VOMNL-2410-14C
 Product group: Walking aid
 Productname: Travixx Classic
 Batchnumber:

Push handle + accessoires

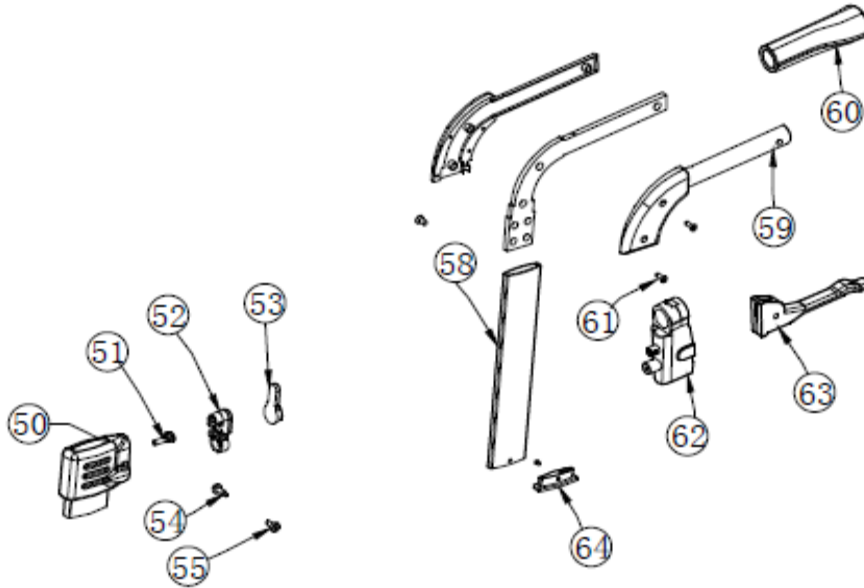

Drawing No.	Item No.	Description	Purchase Price	
50 to 55	92417140	Locking mechanism for handle push bar with grey knob (adjustable)	€	13,76
58 to 64	92417200	Push handle and brake handle (grey) Medium Left	€	19,27
	92417205	Push handle and brake handle (grey) Medium Right	€	19,27
-	92417170	Tray with tray holders (set)	€	43,27
60	92417210	Hand grip for Travixx	€	8,49