


CE-D09-0954-19A Spare parts list



Identification number: TCF-VOMNL-0954-19A
Product group: Power wheelchair
Productname: Excel Qnect
Batchnumber: 4140

Excel Qnect Complete

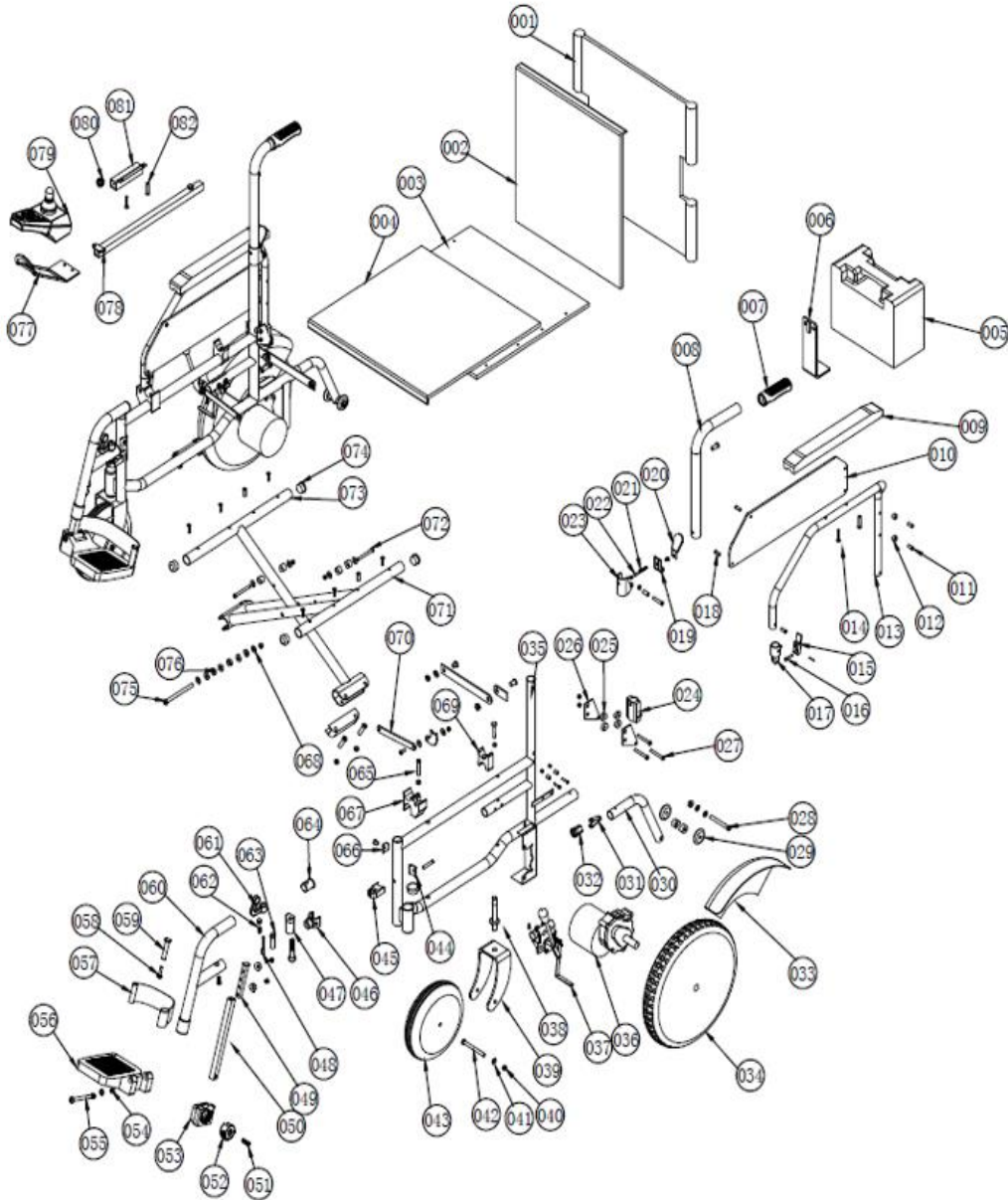

CE-D09-0954-19A Spare parts list



Identification number: TCF-VOMNL-0954-19A
 Product group: Power wheelchair
 Productname: Excel Qnect
 Batchnumber: 4140

Excel Qnect Complete

Drawing No.	Item No.	Description	Price
001	59542040	Backrest 8 tension straps 40 cm	€ 56,61
	59542045	Backrest 8 tension straps 45 cm	€ 56,61
002	59543040	Back upholstery 40 cm	€ 84,91
	59543045	Back upholstery 45 cm	€ 84,91
003	59541040	Seat upholstery 40 cm	€ 47,17
	59541045	Seat upholstery 45 cm	€ 47,17
004	50080438	Seat cushion with Excel logo and anti-slip bottom, seat width 40 cm, seat depth 40 cm	€ 71,36
	50080442	Seat cushion with Excel logo and anti-slip bottom, seat width 45 cm, seat depth 40 cm	€ 71,36
005	19546000	Battery box, incl. battery	€ 415,13
	19546100	Battery box, without battery incl. wires	€ 132,09
	UG330S-P-138-C	Connector for thermal fuse	€ 8,49
	BOT16S-P-188-C	Connector for batteries	€ 9,90
	AJVAN202103006	Cable for charging dock	€ 11,79
006	19546001	Mounting bracket for the battery	€ 7,08
007	99548100	Handgrip LEFT/RIGHT	€ 5,66
008 & 019	19548110	Handle bar (excl. Handgrip) LEFT/RIGHT	€ 25,00
009 & 014	39540111	PU armrest pad LEFT/RIGHT (incl. mounting)	€ 29,72
010 to 012 & 018	39541110	Side panel LEFT (incl. mounting)	€ 28,30
	39541100	Side panel RIGHT (incl. mounting)	€ 28,30
013	39542111	Armrest frame LEFT	€ 21,23
	39542011	Armrest frame RIGHT	€ 21,23
015 to 017	19540450	Release mechanism for flip away armrest FRONT LEFT/RIGHT (consisting of handle, pin and block)	€ 12,50
009 to 018	39540100	Armrest complete LEFT	€ 91,75
	39540110	Armrest complete RIGHT	€ 91,75
020 to 022	19542000	Folding mechanism for backrest LEFT/RIGHT	€ 9,20
024	39540350	Armrest receiving block REAR LEFT/RIGHT	€ 7,08
025 to 027	39542000	Folding mechanism for armrest REAR (set)	€ 20,76
028 to 032	99540015	Anti-tipper complete (set)	€ 46,23
033	49542050	Fender for rear wheel (set)	€ 45,29
034	49542418	Rear wheel LEFT/RIGHT	€ 179,26
034-1	49542420	Hub Left	€ 38,19
034-1	49542425	Hub Right	€ 38,19
023 & 035	19541243	Side frame LEFT/RIGHT	€ 192,47
036	99541990	Motor and transaxle	€ 198,13
037	79540100	Brake assembly LEFT	€ 56,61
	79540110	Brake assembly RIGHT	€ 56,61

CE-D09-0954-19A Spare parts list



Identification number: TCF-VOMNL-0954-19A
 Product group: Power wheelchair
 Productname: Excel Qnect
 Batchnumber: 4140

Excel Qnect Complete

Drawing No.	Item No.	Description	Price
038 & 039	89541000	Front fork complete LEFT/RIGHT	€ 39,15
040 to 043	89542000	Front wheel complete LEFT/RIGHT	€ 59,44
044 & 045	69544720	Footrest fixation block LEFT/RIGHT	€ 25,47
046 to 048 & 061 to 064 & 066	69544250	Revision set for footrest LEFT (consisting of U-shaped handle, cover cap, wishbone and guide block)	€ 36,09
	69544255	Revision set for footrest RIGHT (consisting of U-shaped handle, cover cap, wishbone and guide block)	€ 36,09
049 & 050	69547000	Insert tube of footrest LEFT/RIGHT	€ 23,59
049 to 056	69540040	Footplate incl. insert tube LEFT	€ 74,54
	69540041	Footplate incl. insert tube RIGHT	€ 74,54
057 to 059	69544645	Heel strap incl. mounting (set left & right)	€ 30,66
060	69547100	Hanger footrest LEFT	€ 41,51
	69547110	Hanger footrest RIGHT	€ 41,51
046 to 064 & 066	69541045	Footrest complete sw 40-45 cm LEFT	€ 173,13
	69542045	Footrest complete sw 40-45 cm RIGHT	€ 173,13
065 & 067	19540240	Receiving block armrest FRONT LEFT	€ 9,67
	19540250	Receiving block armrest FRONT RIGHT	€ 9,67
074	29541060	Cover for cross tube (Set of 4)	€ 4,72
068 to 076	29544040	Crossframe complete 40 cm	€ 230,92
	29544045	Crossframe complete 45 cm	€ 230,92
077 & 078, 080 to 082	99549051	Mounting bracket for joystick	€ 97,65
079	99549050	Joystick	€ 367,96
079-1	AJVAN20230801	Cover cap for joystick	€ 19,95
-	93810351	Battery charger 24V/1,8Ah rectangular fitting	€ 224,00